




# Quick Start Manual PLENTY Vaporizer



www.storz-bickel.com

## 1. Open filling chamber

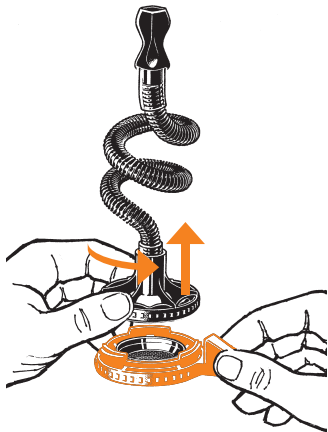

## 2. Grind materials




## 3. Fill filling chamber




Always fill the filling chamber completely up to the upper edge


## Correct Filling

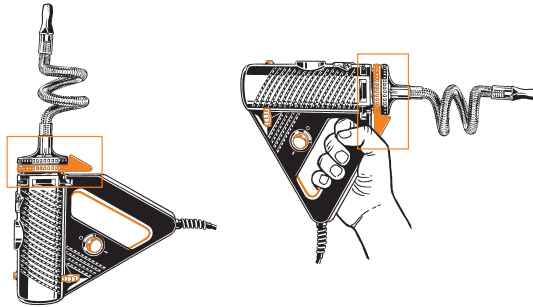

correct

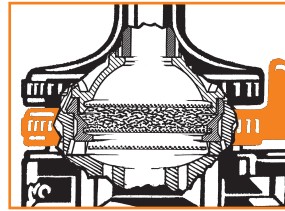


Completely filled filling chamber

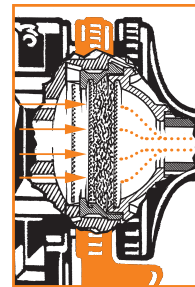


vapor

wrong

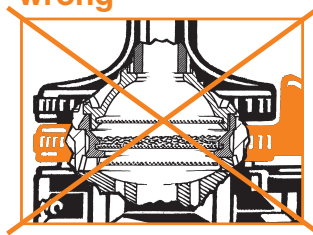


Filling chamber only partially filled

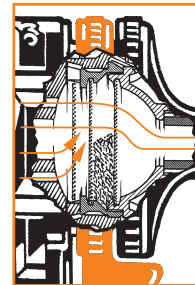


hot air

correct

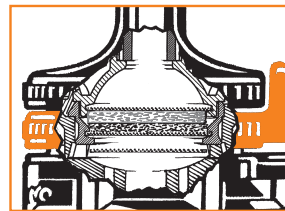


Filling chamber only partially filled, with liquid pad placed on top

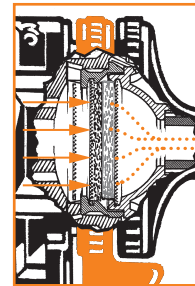


vapor


## 4. Switch on the device (orange power switch) and press auto off release handle.

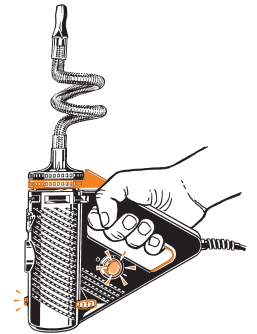

## 5. Air temperature scale

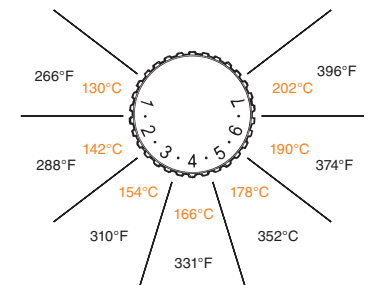

## 6. Choose temperature

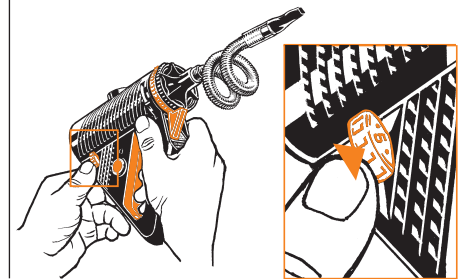

Refer to the Instructions for Use for important safety and user advice. Do not operate the device without first reading and understanding the Instructions for Use!

STORZ & BICKEL GMBH & CO. KG · Rote Strasse 1 · 78532 Tuttlingen/Germany · Telefon: +49-7461-969707-0 · Fax +49-7461-969707-7 · info@storz-bickel.com  
STORZ & BICKEL AMERICA, INC. · 1078 60<sup>th</sup> Street, Suite A · Oakland · CA 94608 · USA · phone 510-451-1553 · fax 510-451-2053 · oakland@storz-bickel.com




## Disassembling, Cleaning, Assembling



www.storz-bickel.com


### 1. Expanded view of the PLENTY Vaporization Unit

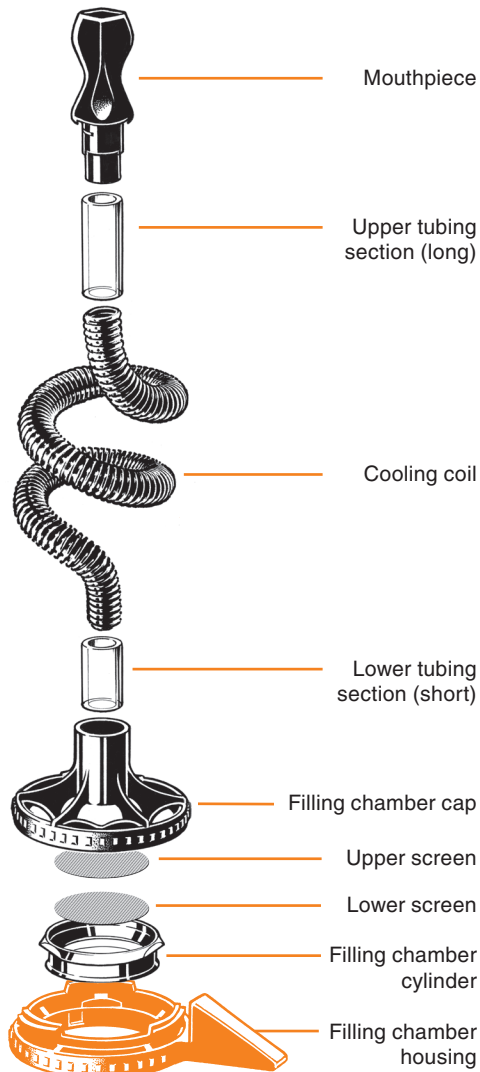


Free  
Shipping  
Worldwide




### 2. Pull the mouthpiece away from the cooling coil

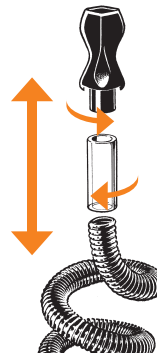


### 3. Pull the cooling coil out of the filling chamber cap

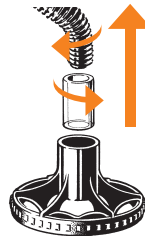


### 4. Push out the upper screen with help of the cleaning brush

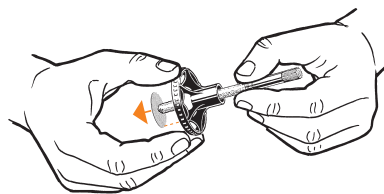


### 5. Disassembling the filling chamber

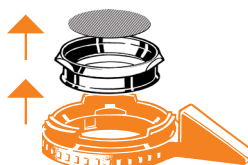


### 6. Push out the filling chamber cylinder

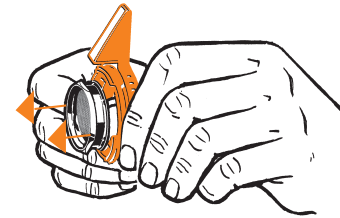

### 7. Push out the lower screen

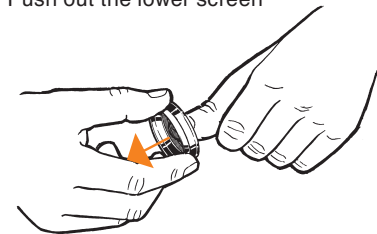

### 8. Manual cleaning with alcohol (min. 90%) or dishwashing soap

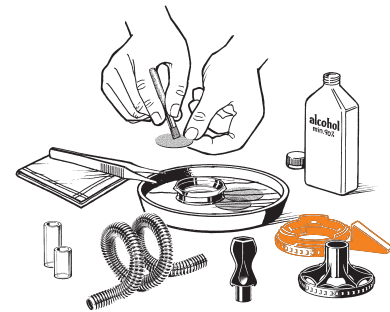

### 9. Filling chamber cylinder, filling chamber housing, positioning surfaces

